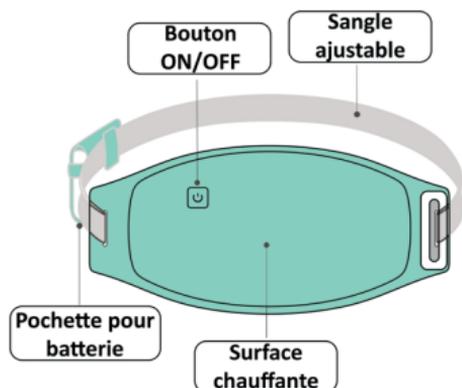
Ceinture Menstruelle (P2)

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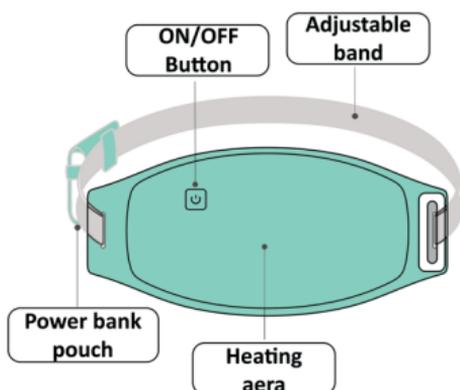
User manual

Version with 5 000 mAh power bank
(K1-115570)

3 temperature levels: Fast, healthy and penetrating far-infrared heat.

Red light: 65°C / 150°F

Green light: 55°C / 130°F

Blue light: 45° / 115°F

Installation:

1. Place and adjust the belt with the battery pouch around your waist.
2. Slide the battery pouch to the right of the heating pad, on the USB cable side.
3. Connect the belt to a power source (power bank, computer...).
4. For back use, it is advised to slide the belt around you once the belt is in place.

Installation
in video
+ bonus

Version with power bank included: connect the belt's USB cable to one of the two USB ports «OUT» on the power bank.

With a power bank, you can keep on moving while wearing the belt.

Use:

1. To turn on the belt, press the ON/OFF button for 2 seconds.
The belt will default to its hottest temperature (red light - 65°C/150°F).
This temperature will be maintained for 5 minutes, then the belt will automatically switch to 55°C (green light - 130°F).
2. Short press to change the temperature.
3. To turn off the belt, press the ON/OFF button for 2 seconds.

The belt will automatically turn off after 45 minutes, regardless of the selected temperature.

Version with power bank included: the battery turns on and off automatically.
If necessary, press and hold for 10 seconds to turn it on or off.

Safety:

1. Disconnect the USB cable from the battery when not in use.
2. For your safety, turn off the belt before going to bed.
3. Do not use on damaged skin. Make sure the skin supports the belt.
If discomfort occurs, remove immediately.
Consult a health care professional if necessary.
4. Sweat resistant, but do not allow prolonged contact with any liquid.
5. Unplug the belt before cleaning.
After cleaning, do not use until the belt is completely dry.
6. Hand wash only, gently with a little soapy water. Rinse well. Do not submerge.
Do not wring out - allow to air dry.
7. Do not fold the heating pad or place heavy objects on it.
8. Do not cut or pierce. Do not use the product if it is damaged.