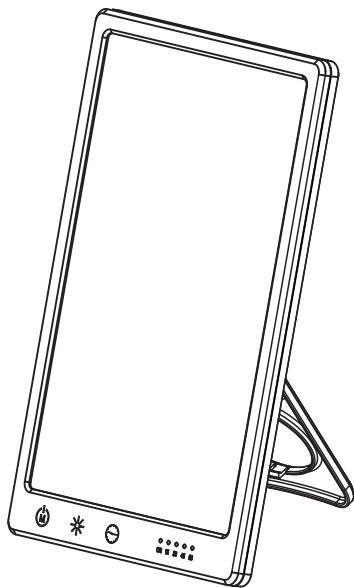
# Light Therapy Lamp

ML-L001

English..... P1~6

Français..... P7~12

Deutsch..... P13~18

Italiano..... P19~24

Español..... P25~30

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# Light Therapy Lamp

## ML-L001

## User Manual

Touch Button  
Switch / Mode  
Circulation Button  
ON  
Warm Yellow Mode 3000K  
Warm White Mode 4250K  
Cool White Mode 6500K

Touch Button  
Timer Button  
Circulation Button  
15 Min  
30 Min  
45 Min  
60 Min

Touch Button  
Brightness Button  
Circulation Button  
Brightness Level 1 : 3000lux  
Brightness Level 2 : 8000lux  
Brightness Level 3 : 10000lux  
Brightness Level 4 : 15000lux

### Indicators:

ON: Power on, showing in green.  
Steady on, showing in blue.

15: Indicates 15 minutes timer, showing in blue;  
30: Indicates 30 minutes timer, showing in blue;  
45: Indicates 45 minutes timer, showing in blue;  
60: Indicates 60 minutes timer, showing in blue.

## **Introduction**

Light therapy lamp can help you fight the SAD (seasonal affective disorder) and combats winter blue by simulating natural daylight.

## **What is SAD?**

Seasonal affective (SAD) is a mood disorder, a type of major depressive disorder, and sufferers may exhibit any of the associated symptoms, such as feelings of hopelessness and worthlessness, thoughts of suicide, loss of interest in activities, withdrawal from social interaction, sleep and appetite problems, difficulty with concentrating and making decisions.

Light therapy is the leading treatment for seasonal affective disorder.

## **Benefits:**

- ① Simulates natural daylight
- ② Improves physical & mental wellbeing during dark short winter days
- ③ Combats the effect of jet lag
- ④ Help the treatment of seasonal affective disorder (SAD)
- ⑤ Bring back energy to fight winter blue

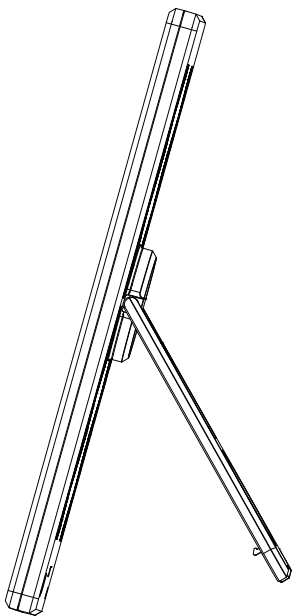
## Functions & Operations:

- ① Slightly touch the “M” switch button, the screen lights up; keep touching this button, the color temperature will circulate among “ON – warm yellow – warm white – cool white – OFF” in turns. Turn off the lamp and restart it, it will show warm yellow light again.
- ② Slightly touch the brightness button, it will switch among brightness level 1/2/3/4 in turns. The brightness will increase 25% when switching to the next level; turn off the lamp and restart it, the brightness will be back to level 1.  
The brightness will remain when switching the color temperature.
- ③ Slightly touch the timer button to turn on the timers of 15Min - 30Min - 45Min - 60Min – OFF.

## Display:

- ① When it is power on, the lamp is standby, the “ON” indicator is in green;
- ② Slightly touch the “M” button to start up the lamp, the “ON” indicator is in blue;
- ③ When the lamp is on, adjust the brightness, no indicator show;

- ④ Slightly touch the timer button, the corresponding indicator is in blue at figure 15/30/45/60, showing timers of 15min, 30min, 45min, or 60min.

The Back Support  
Max. Angle: 45 degree

## Technical Parameters:

Product Type	ML-L001
Input	5V/3.5A
Power	2.5-13W
USB Port	Type-C
Brightness	3000-16000LM
Material	PC+ABS
Size	226*145*18.3mm
Weight	433g

## Packing List:

Light Therapy Lamp\*1      Type-C Charging Cord\*1  
User Manual\*1              Adapter\*1

## Attentions:

- ❶ Please do not squeeze or disassemble, or place the lamp in high temperature (60℃ or above) for a long time, otherwise it will be damaged.
- ❷ Please do not clean the lamp with hard metal or corrosive liquid. Please do not disassemble the device.
- ❸ Please store the lamp out of the reach of children. The lamp should be used by children under the supervision of adults.
- ❹ Please do not use the lamp in humid environment.

## Warranty card:

Dear customer, thanks you for purchasing our company's products.  
Please read the instruction manual and save it for future reference.  
Please fill out the warranty card and place it in the mail.

Customer name:
Address:
Date of purchase:
Phone number:
Email:
Product model#:

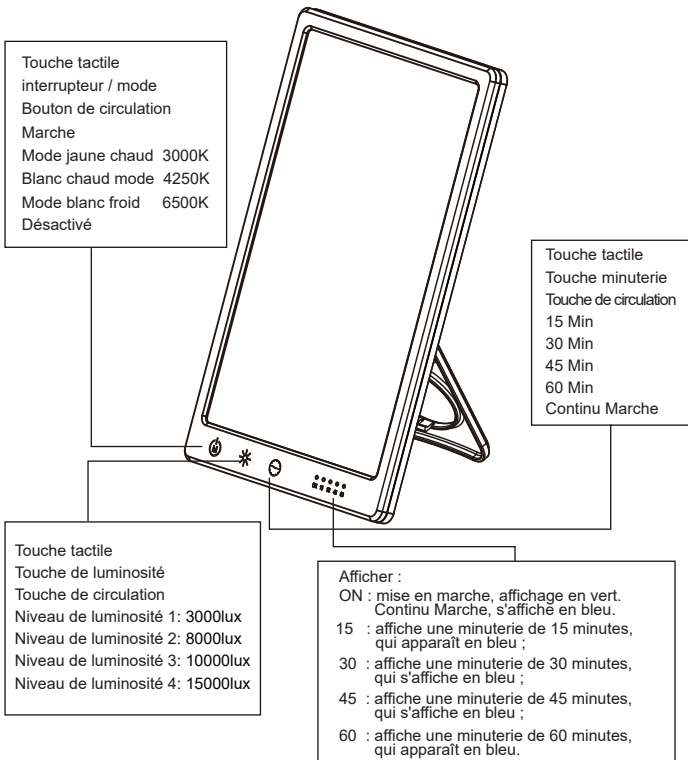
Manufacturer: Dongguan V-FLAG Electronic Technology CO., LTD  
Address : 901, Building 3, Zhongli Hi-Tech Industrial Park, No.1,  
Guliao 2nd Road, Tangxia Town, Dongguan City,  
Guangdong Province, China

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