luminette® Drive

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USB tipo C

USB typu C

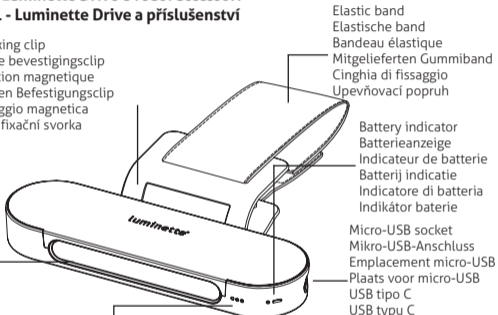
Figure 2: Choice of intensity versus distance

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EN

I. Introduction

You have just purchased the Luminette® Drive. We would like to congratulate you on your purchase and welcome you to the world of on-the-go light therapy!

Important: Before turning on and using your Luminette® Drive, please carefully read this manual and be sure to keep it in a safe place, also certain that you take enough time to get to know your device using Figure 1.

A. The role of light

By means of the light/dark cycle, the human brain synchronizes functions such as mood, appetite, digestion, sleep, libido, etc. These circadian rhythms, which oscillate over 24 hours, constitute our biological clock.

Melatonin is what is commonly referred to as the "sleep hormone". The brain secretes it at nightfall, which then induces sleep. Conversely, at daybreak, in the presence of light, the brain stops the secretion of melatonin, enabling us to get up easily, and full of energy.

In addition to its regulating effect on our biological clock, light significantly affects mood, alertness, and physical and intellectual performance, by improving them.

B. Light therapy

In some situations, the biological clock is no longer in sync with external environments. The circadian rhythms are, therefore, "desynchronized". This is particularly the case in winter when there is an insufficient amount

of light, at which point some people develop winter blues. This is also something that can be seen in people who tend to go to sleep too early or too late.

Jet lag is another type of disruption to the biological clock, as well as fragmented work disorders (which affect more than 10% of workers in Europe) which are now recognized as real pathologies. All these disorders can be reduced or even eliminated by the correct use of light and darkness. When light cannot be found in natural form, light therapy is the solution that offers the most benefits with the least side effects.

C. Its applications

- Winter Blues
- Fatigue - low energy
- Problems with sleep rhythms
- Work with a fragmented schedule

This product is not approved as a medical device under 93/42/CEE norms.

D. Contraindications

Whatever the reason for your purchasing Luminette® Drive, we advise that you avoid exceeding the recommended daily usage.

There are few contraindications.

If you suffer from an eye condition (glaucoma, retinis, retinopathy, macular degeneration), we recommend that you consult a specialist before

using Luminette® Drive. If you have cataracts, light therapy may be less effective.

If you suffer from psychiatric disorders (depression, hysteria, neurosis, and psychosis), we recommend that you consult a doctor before using this device.

Luminette® Drive can also be used with contact lenses or corrective lenses.

E. Possibles reactions

Adverse reactions from the light of this device are relatively mild and temporary. They may include:

- Headaches.
- Eye strain or visual disturbances, intraocular pressure.
- Nausea.
- Agitation, sedation, nervousness, and very rarely hypomania or mania.

In order to avoid, reduce, or mitigate these adverse effects, the following measures can be taken:

- Increase the brightness inside your vehicle.
- Reduce the light intensity of the device if you have any of these reactions.
- Avoid using the device for a few days until the adverse reactions have disappeared, then try again.

F. Safety

This equipment complies with the technical regulations and safety standards applicable to electric devices and related to exposure to electromagnetic fields (EMF). This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. To ensure the longevity of your Luminette® Drive, we advise you to:

- Only use the original accessories.
- Never immerse the Luminette® Drive in water.
- Do not wash your Luminette® Drive in a lot of water.
- Do not leave your Luminette® Drive in direct sunlight, or damp or dusty places.

Caution!

- No modifications to the device and its accessories are permitted.
- This unit cannot be connected to an extension cord or power strip.
- Always use the unit with the adapter supplied or any other compatible system as specified in the manual.
- The adapter is the network disconnect device and must always remain easily accessible.
- The outer shell of the main plastic shell can reach temperatures of 49°C.

- If there is any interference, electromagnetic, or other, then move the Luminette device away from the device that is interfering with it.
- The Luminette device should not be used next to or on top of other devices because it may cause a malfunction. In the event that such usage is unavoidable, monitor both devices and make sure of their normal operation.
- The use of accessories and cables other than those specified or provided may cause an increase in electromagnetic emissions or a decrease in the integrity of the equipment and may result in improper operation.
- Portable radio frequency communications devices (including items such as antenna cables and external antennas) should not be used closer than 30 cm (12 inches) to any part of the Luminette® Drive, including specified cables. Otherwise, the performance of these devices may be affected.

G. Road safety

The user shall remain solely responsible for the use of the device at all times and in strict compliance with the terms of service and the Highway Traffic Act.

In particular:

- Device settings (brightness or beam angle) may only be adjusted when the vehicle is stationary and off the road.
- Luminette Drive is not a substitute or complement to roadway lights.

Only use the device during the day or on sufficiently lit roads, in good light and visibility conditions.

- Immediately discontinue use of the device in the event of glare, reduced field of vision, or discomfort.

LUCIMED is not responsible for accidents

II. Use

A. When can I use Luminette® Drive?

1. To combat seasonal blues

Lacking energy and motivation? Have difficulty getting up in the morning? Feel the need to hibernate? Snack more? Notice a decrease in your libido? You are probably experiencing the winter blues. This condition is the result of a reduction in brightness in autumn and winter. We recommend a light therapy session as soon as possible after waking up.

2. To combat fatigue and low energy

The Luminette® Drive reproduces the beneficial effects of the sun and stimulates specific receptors in the eyes that trigger the body's energizing response to light. This helps combat energy loss, makes you feel more fit, active, and alert, it can also improve your mood. We recommend a light therapy session as soon as possible after waking up.

3. To combat sleep rhythm disorders

Some sleep disorders are caused by an advance or delay in the biological clock. In these cases, light therapy can provide a solution.

- A delay in the biological clock

Have a tendency to fall asleep too late and have difficulty getting up in the morning? We recommend you have a light therapy session as soon as possible after waking up.

- An advance in the biological clock

Have trouble staying awake in the evening and wake up too early? We recommend that you have a light therapy session in the late afternoon or early evening, two to three hours before the time you want to go to bed.

4. To optimize fragmented work

If you work at night for more than three consecutive days and have difficulty staying awake at work and/or on the way home, we recommend you have a light therapy session just before starting work and, in case of drowsiness, a second session during your work break.

As a precaution, we advise you to contact your occupational physician.

B. How long does a luminotherapy session last?

The recommended duration to use your Luminette Drive is 20 minutes. Once the session is finished, the device turns off automatically.

C. How often should I use the Luminette® Drive?

It is recommended that you use the Luminette® Drive once per day.

D. How long before you can feel the effects?

Some people start to feel the regulating effects of Luminette® Drive right from the first use. Others, after one to two weeks of daily use.

E. How do you charge the Luminette® Drive?

Charging the device is very simple.

To do this:

- Connect the type-C connector of the cable to the specified location on your Luminette® Drive (see figure 1) and connect the other standard USB connector to the charger port.
- Plug the charger into the cigarette lighter port of your car.
- When the device is fully charged (charging time is about 2 hours), disconnect the cable from the device.

Charge indication:

- When your Luminette® Drive is charging, the green charging indicator light flashes slowly.
- When your Luminette® Drive is fully charged, the same green light stops flashing and remains On.
- In the event of an error (battery or circuit failure), the red LED flashes quickly.
- The device can be used while it is charging.

Device autonomy:

Its autonomy depends on the intensity chosen. As an indication, the Luminette® Drive should have an autonomy of at least 5 sessions. The remaining battery life is indicated by the indicator light: it is green when the battery life of the device is sufficient for one session and red when it is recommended you recharge the device.

F. Turning On/Off

Caution: Before the first use, make sure the device is fully charged to allow your Luminette® Drive to operate for the entire duration of the session.

To turn on your Luminette® Drive, press and hold the On/Off button briefly (see figure 1). To change the intensity of the light, simply press the same button briefly. The change is made cyclically: level 1, level 2, level 3, then a return to level 1, etc. The Luminette® Drive stays on for 20 minutes, blinks three times when the session is finished and turns off automatically. To turn off your Luminette® Drive in the middle of a session, hold down the On-Off button (for 2 seconds).

G. How do you position the Luminette® Drive?

For the maximum efficiency and ideal driving comfort, you must place the Luminette® Drive correctly in your vehicle.

To properly position your Luminette® Drive:

- Place the magnetic fixing clip on your sun visor with the elastic band provided and tighten it firmly.

- “Clip” the device onto your sun visor using the magnetic clip.
- Using a tape measure, measure the distance between your face and the device in order to determine the appropriate light intensity.
 - If your face is less than 10 cm from the device, choose intensity n°1.
 - If your face is between 10 and 20 cm from the device, choose intensity n°2.
 - If your face is more than 20 cm from the device, choose intensity n°3.

When looking at yourself in the central mirror of your car, make sure that the blue light band is positioned across your eyes.

Caution: Be careful not to look directly into the LED.

H. How to maintain the Luminette® Drive?

This device is made for use in cars. After each use, it is highly recommended that you unhook the device and its magnetic housing and store it in an appropriate place (glove compartment, for example). To clean your Luminette® Drive, use a soft damp cloth that has been dipped in clear water and soft cleaning solution (denatured alcohol). Be sure to protect also the fragile illuminating lens, and avoid putting your fingers on it. If there are still marks on it despite your precautions, simply clean them with cotton.

IV. Guarantee and after-sales service

A. Guarantee

Lucimed guarantees the operation of its equipment in accordance with the instructions for use and against any manufacturing defect. This warranty covers parts and labor. In the event of device failure, Lucimed will provide free repair for 2 years starting from the date of purchase. The warranty is only valid upon presentation of the purchase invoice. The date of intervention must be within the warranty period. The warranty does not cover faults due to accidental causes, misuse, or negligence, and in the event modifications or repairs have been carried out by unqualified persons. Under no circumstances may this guarantee give rise to any claim for damages. Lucimed provides after-sales service during and after the warranty period in all countries where the product is officially sold and distributed. The consumer benefits from the conditions of articles 1641 et according to the Civil Code relating to the legal guarantee. In all correspondence, mention the serial number and date of purchase. In case of problems, go to the website. Caution! If you lose this instruction manual, please remember that it can be downloaded from www.myluminette.com.

B. Accessories

The following accessories are available when purchasing:

- C type adapter and charging cable: reference SN-189N-2U-2.4A-C2

V. Environment

If your device stops working, do not throw it away with the domestic waste. The integrated rechargeable battery in the device contains substances that are harmful to the environment. Please remove the battery before throwing the device away, and take it to a designated area for that purpose. This will help to protect the environment. If you are unable to remove it, bring the device back to where you bought it.

To remove the battery:

- Make sure that the device is disconnected and the battery is empty.
- Unscrew the cover of the unit using a suitable tool.
- Disconnect the battery cable from the battery and remove the battery from the unit.

VI. Technical specifications

A. Characteristics of the device

General	
Registered trademark	Luminette®
Device model	Luminette Drive
Adapter model	SN-189N-2U-2.4A-C2
Electricity	
Nominal input voltage of the adapter	12-24V DC
Nominal input frequency of the adapter	50-60 Hz
Nominal output voltage of the adapter	5V DC
Nominal output power of the adapter	12 W
Power type	
1 Rechargeable lithium ion batterie	3.7V 2200 mAh
Classification	
Operation mode	Continuous
Insulation class in operation	Internal power source (battery)

Insulation class in recharge mode	II
Protection class	IP20
Physical specifications	
Dimensions	18 x 5,5 x 2,5 cm (7 x 2.2 x 1 inch.)
Weight	155 gr (5.47 oz.)
Light intensity in position 1 to 10cm	± 100 lux
Light intensity in position 2 à 20 cm	± 100 lux
Light intensity in position 3 à 30 cm	± 100 lux
Spectral specifications	
Peak Wavelength Bandwidth	470 nm 50 nm
Nominal Spectral irradiance Intensity 1 at 10 cm, 60° Intensity 3 at 30 cm, 30°	49 $\mu\text{W}/\text{cm}^2$ 65 $\mu\text{W}/\text{cm}^2$
Operating conditions	
Temperature	0° C – + 45° C

Relative humidity level	35 % – 85 %
Atmospheric pressure limits	700 hPa - 1060 hPa
Storage and transport conditions	
Temperature	-20° C – + 40° C
Relative humidity level	35 % – 85 %
Life expectancy	
Device lifetime	6 years
Adapter lifetime	3 years

B. Symbols

The following symbols may appear on the device:

Symbols	Description
	Made for: Lucimed SA, Rue le marais 12a, 4530 Villers-le-Bouillet, Belgium Tel: +32 4 369 48 36
	Year of manufacture

 SN 0000	Serial number
	Refer to the instructions provided.
	This equipment must be disposed of in an appropriate way and recycling facility.
	Read the user manual carefully before use.
IP20	Degree of protection against ingress of foreign bodies and water
	Keep dry
	This device's packaging is recyclable
CE	CE stands for European Declaration of Conformity.

C. Manufacturer's responsibility

As the manufacturer, we are responsible for the safety, reliability, and operation of the device only when it is used in accordance with the operating instructions and when modification and repairs have been carried out solely by personnel authorized by us. Please contact the manufacturer if you require assistance with the installation, use or maintenance of the unit or if you encounter any unexpected problems. Any serious incident related to the Luminette® Drive must be reported to Lucimed and the competent authority in your country.

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